



## In Clinic Titer Testing for Core Vaccines:

- **Canine Parvo Virus, Distemper and Adenovirus Virus**
- **Feline Panleukopenia, Calici and Herpes Virus**





## Approved By:



ISO 13485:2003 Medical devices



USDA & Canadian Food Inspection Agency



Japanese Ministry of Agriculture, Food and Fisheries

# VacciCheck Kit

A reliable and versatile IgG antibody dot-ELISA based system for the confirmation of core vaccines immunity in dogs and cats, recommended by WSAVA and AAHA.


Available kits:

12 tests

120 tests



# Reading the results:

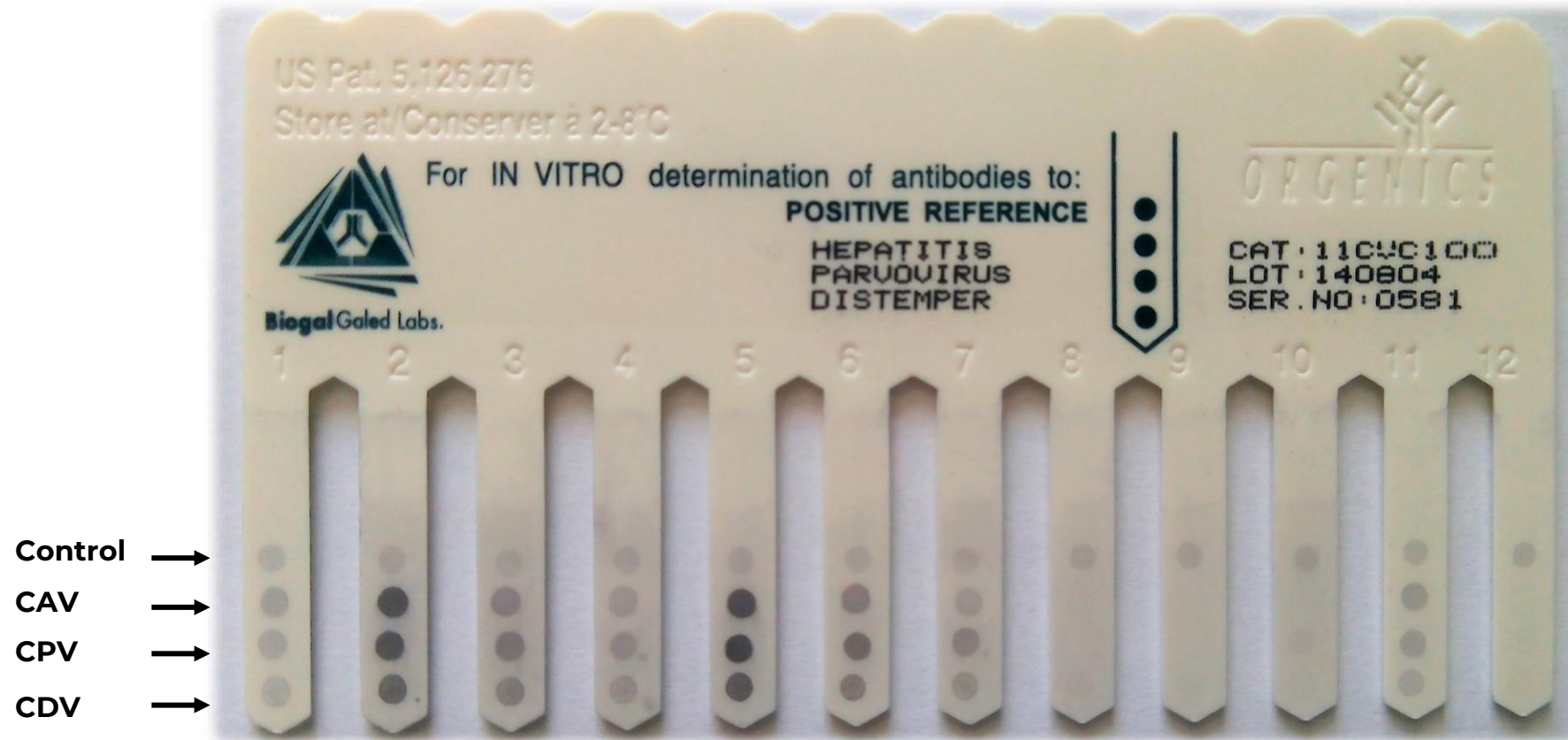

# What VacciCheck titer scores means:

STEP 4		RESULTS INTERPRETATION	
SCORE	RESULT	IMMUNITY STATUS	RECOMMENDATIONS
S0 - S1	Negative	Poor immunity	Vaccination Required
S2	Weak Positive	Adequate Immunity*	The dog will not benefit from repeated vaccination**
S3 - S4	Positive	Good Immunity	
S5 - S6	High Positive	Excellent Immunity	

## For core vaccine immunity monitoring

\*According to the 2016 WSAVA Guidelines.

\*\*“The presence of antibody (no matter what the titre) indicates protective immunity, and immunological memory is present in that animal”

\*\* Giving more frequent vaccines to animals in an attempt to increase antibody titre is a pointless exercise. It is impossible to create ‘greater immunity’ by attempting to increase an antibody titre.